## Gebrauchsinformation » Tray adhesive for VPS«

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## Instructions for use » Tray adhesive for VPS«

*Tray adhesive for VPS* is a tray adhesive for use when taking impressions using metal and plastic trays.

### Indication

*Tray adhesive for VPS* should be used with impression trays that have no or only minimal retention. The tray adhesive provides reliable retention of addition-cured silicone to metal and plastic trays. When using plastic trays, ensure that the impression tray is compatible with the adhesive.

### Recommended use

1. The impression tray must be clean, grease free and dry.
2. Brush the inside of the tray with *Tray adhesive for VPS* so that there is a uniform, thin film.
3. Allow the adhesive to react for 3:00 minutes to ensure a good bond to the tray.
4. Load impression material into the tray within 30 minutes so that the adhesive produces its full bond strength.

### Please note

- Flammable
- Do not inhale
- Do not swallow
- Avoid contact with the skin and eyes
- Avoid liquid coming into contact with the skin. If there is accidental contact, wash with soap and water. If there is contact with the eyes, rinse immediately with plenty of water and consult a doctor.

### Contraindications

The application of *Tray adhesive for VPS* is contraindicated when allergies against any components are known.

### Side effects

Side effects are not known to date. In singular cases, hypersensitivity or contact allergy to components of the material cannot be excluded.

### Composition

Silicone resin, solvent (volatile paraffin).

### Storage

Store at normal room temperature ( $\leq 25\text{ °C}$  /  $\leq 77\text{ °F}$ ). Please close bottle immediately after use to prevent contamination as well as volatilization of the solvent. Do not use after date of expiry.

### Available product sizes

#### Tray adhesive for VPS

REF 9046 1 bottle @ 10 ml

**Keep away from children! For dental use only!**