»VPS Medium/VPS Medium (fast)«

de

Produktbeschreibung

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Das Produkt entspricht ISO 4823:2021, medium bodied consistency

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en-US

Product description

VPS Medium/VPS Medium (fast) is a medium consistency precision impression material based on VPS addition-curing silicones and is automatically mixed.

Indications

Dental cases that require the production of dental models.

Contraindications

Do not use the material where there are contact allergies or known allergies to any of the components.

Intended use

Impressions taken using the correction impression technique in conjunction with *VPS Putty* or monophasic impressions for the fabrication of crowns, bridges, inlays and onlays, for the determination of pressure points or for functional and relining impressions.

Limitations of use

There are no known limitations of use.

Intended users

Dentist, dental assistant staff

Patient target group

Persons receiving treatment as part of a dental procedure.

Recommended use

Preparing the impression

Have *VPS Medium/VPS Medium (fast)* correction impression material ready for use, and shortly before application, check that the openings of the safety cartridge are not obstructed. The used mixing tip acts as a seal until the next application is carried out. Do not reuse the original closure cap of the safety cartridge under any circumstances! This can result in unwanted curing due to contamination of the base and catalyst pastes. If retraction fluids have been used, carefully rinse and dry the region prior to taking the impression.

Corrective impression

Place the impression tray filled with *VPS Medium/VPS Medium (fast)* in the mouth. This position should be maintained for at least as long as the specified setting time in the mouth for the material. After removal, thoroughly clean and dry the initial impression. Prior to performing the correction impression, cut drainage channels into the initial impression and eliminate any defects.

Place *VPS Light/VPS Light (fast)* correction impression material into the initial impression and if necessary, inject the material around the prepared teeth. Place the filled correction impression back into the mouth, observing the working and setting time of the correction impression material. To ensure that the two impression materials bond securely together, do not wait longer than 30 min between the completion of the initial impression and preparation of the final impression.

Monophase impressions

Place the impression tray filled with *VPS Medium/VPS Medium (fast)* in the mouth. This position should be maintained for at least as long as the specified setting time in the mouth for the material. After taking the impression, clean the impression thoroughly and rinse under running water for approx. 15 seconds. Dry the impression carefully to obtain a bubble-free model.

Disinfection and casting

The impression can be disinfected using commercially available dipping solutions. The use of a 2 % glutaraldehyde solution is recommended. Please ensure that the disinfectant used is indicated for use with addition-cured silicone impression materials.

The exposure period for the impression is approx. 10-15 min. Afterwards, rinse the impression under running water for 15 s. Store the impression at normal room temperature (max. 25 °C/77 °F). Wait at least 30 min after removing the impression from the patient's mouth before casting.*

The impression material is compatible with all commercially available plasters. The use of a hard plaster (ISO 6873:2013, Type 3, e.g. Moldano, Bayer) for the cast and an ultra-hard dental plaster for the molds (ISO 6873:2013, Type 4, e.g. Fujirock II, GC) is recommended. The impressions can be copper or silver-plated.

* Additional information: Statement by the DGZMK 3/93 V 1.0; yiryim.dgzmk.de
ADA recommends following the advice of the Council on Dental Materials, Instruments and Equipment. Disinfection of impressions.
JADA1991;122(3):110; wym.ada.org

Storing the impression

- Store the impression at 15–25 °C (59–77 °F) in a dry place!
- Protect against exposure to direct sunlight!
- The impression is stable and can be cast even after months without loss of accuracy

Timing

	VPS Medium	VPS Medium Fast
Working time (from the start of mixing)*	≤ 135 s	≤ 90 s
Recommended setting time in the mouth	≥ 210 s	≥ 135 s

* The specified working times are valid for a room temperature of 23 °C (72 °F) and a normal relative humidity of 50%. Higher temperatures shorten, lower temperatures lengthen these times.
The working time describes the period after mixing has begun until the time by which the impression tray must be positioned in the patient's mouth.

Notes for use

- The recommended times can be found in the section headed »Timing«.
- If the material is applied to the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.
- Non-perforated metal or plastic impression trays can be used with *VPS Medium/VPS Medium (fast)*.
- When selecting the impression tray, ensure that there is enough room between the wall of the impression tray and the dental arch.

»VPS Medium/VPS Medium (fast)«

- In order to improve the adhesion of *VPS Medium/VPS Medium (fast)* to non-perforated impression trays, apply a thin, even film of *Tray adhesive for VPS*. Allow *Tray adhesive for VPS* to dry for 3 min before loading the impression tray.

Interactions

- Do not use with condensation-curing or polyether impression materials.
- Do not allow the impression to come into contact with organic solvents or liquids that contain solvents as this can cause the material to swell and lead to inaccuracies.
- As retraction solutions may impair the setting time of the material, any retraction solution residues must be removed prior to taking the impression by thoroughly spraying and drying the region.
- Do not use with latex gloves because they inhibit the setting process. The use of PVC or PE-based gloves is recommended.

Cartridge handling

The double syringes and cartridges enable the automatic dispensing and mixing, which allows a time-saving direct application. When using a new cartridge, remove the transport cap first. The mixing tip and, if necessary, an intraoral tip can then be fitted. After use and until the next treatment, leave the mixing tip on the cartridge as a seal. The used mixing tip is replaced with a new tip when the device is used for the next treatment.

Note: When using a double syringe or cartridge for the first time, discard a small amount of the initially ejected material (e.g. a pea-sized amount).

Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

Warnings/precautions

- Keep out of the reach of children!
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Product performance characteristics

The product complies with ISO 4823:2021, medium bodied consistency

Storage/disposal

- Store in a dry place at 15 to 25 °C (59 to 77 °F).
- Do not use after the expiration date.
- Disposal must comply with national regulations.

Composition

REF 9050MDR: Vinyl silicones/siloxanes, methyl silicones/siloxanes, SiO₂, additives, pigments, catalysts, hydrogen silicone.

REF 9051MDR: Vinyl silicones/siloxanes, methyl silicones/siloxanes, SiO₂, additives, pigments, catalysts, hydrogen silicone

Packaging

VPS Medium	
REF 9050MDR	2 Cartridges @ 50 ml 12 Mixing Tips 12 Intraoral-Tips
VPS Medium (fast)	
REF 9051MDR	2 Cartridges @ 50 ml 12 Mixing Tips 12 Intraoral-Tips