»VPS Situation«

de

Produktbeschreibung

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en-US

Product description

VPS Situation is a medium consistency precision impression material based on VPS addition-curing silicones and is automatically mixed. The material is suitable for all kinds of anatomical impressions.

Indication

Dental cases that require the production of situation models.

Contraindication

Do not use the material where there are contact allergies or known allergies to any of the components.

Intended purpose

- Impressions with fully dentulous, partially dentulous or edentulous jaws for making situation models
- Impressions for the fabrication of temporary crowns and bridges
- Impressions for the fabrication of thermoformed splints
- Orthodontic work
- Opposing jaw models

Limitations of use

There are no known limitations of use.

Intended user

Dentist, dental assistant staff

Patient target group

People treated in the course of a dental procedure.

Recommended use

Monophase impressions

Load the impression tray with *VPS Situation* and place the tray into the patient's mouth. Maintain this position for at least the recommended setting time in the mouth for the material. After taking the impression, clean the impression thoroughly and rinse under running water for approx. 15 s. Dry the impression carefully to obtain a bubble-free model.

Disinfection and casting

The impression can be disinfected using commercially available dipping solutions. The use of a 2 % glutaraldehyde solution is recommended. Please ensure that the disinfectant used is indicated for use with addition-cured silicone impression materials.

The exposure time for the impression is approx. 10 to 15 min. Afterwards, rinse the impression under running water for 15 s. Store the impression at normal room temperature (max. 25 °C/77 °F). Wait at least 30 min after removing the impression from the patient's mouth before casting.*

The impression material is compatible with all commercially available plasters. The use of a hard plaster (ISO 6873:2013, Type 3, e.g., Moldano, Bayer) for the cast and an ultra-hard dental plaster for the molds (ISO 6873:2013, Type 4, e.g., Fujirock II, GC) is recommended. The impressions can be copper or silver-plated.

* Additional information: Statement by the DGZMK 3/93 V 1.0; yiry.dgzmk.de
ADA recommends following the advice of the Council on Dental Materials, Instruments and Equipment. Disinfection of impressions.
JADA1991;122(3):110; wym.ada.org

Storing the impression

- Store the impression 15 – 25 °C (59 – 77 °F) in a dry place!
- Protect against exposure to direct sunlight!
- The impression is stable and can be cast even after months without loss of accuracy.

Timing

Working time (from the start of mixing)*	≤ 75 s
Recommended setting time in the mouth	≥ 105 s

* The specified working times are valid for a room temperature of 23 °C (72 °F) and a normal relative humidity of 50 %. Higher temperatures shorten, lower temperatures lengthen these times.
The working time describes the period after mixing has begun until the time by which the impression tray must be positioned in the patient's mouth.

Notes for use

- The recommended times can be found in the section headed »Timing«.
- Perforated or non-perforated metal or plastic trays can be used with *VPS Situation*. When selecting the impression tray, ensure that there is enough room between the wall of the impression tray and the dental arch.
- In order to improve the adhesion of *VPS Situation* to non-perforated impression trays, apply a thin, even film of *Tray adhesive for VPS*. Allow *Tray adhesive for VPS* to dry for 3 min before loading the impression tray.

Interactions

- Do not use with condensation-curing or polyether impression materials.
- Do not allow the impression to come into contact with organic solvents or liquids that contain solvents as this can cause the material to swell and lead to inaccuracies.
- As retraction solutions may impair the setting time of the material, any retraction solution residues must be removed prior to taking the impression by thoroughly spraying and drying the region.
- Do not use with latex gloves because they inhibit the setting process. The use of PVC or PE-based gloves is recommended.

Cartridge handling


Insert the cartridge into the mixing device. Please refer to the operating instructions of the respective device manufacturer.

Attaching / changing the mixing tip


Open the locking clip by pulling it along the track to its terminal position and pull off the original closure cap or the old mixing tip. (Fig 1-2)

»VPS Situation«

1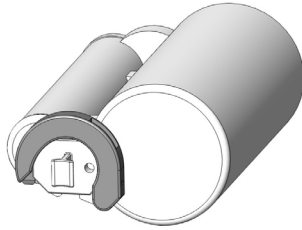


2

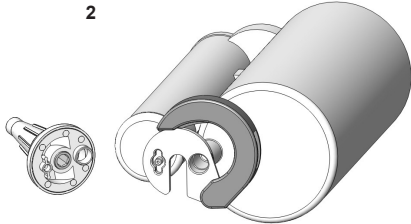
Then insert a new mixing tip and press both openings on the back of the mixing tip firmly onto the corresponding openings of the cartridge. (Fig 3)

3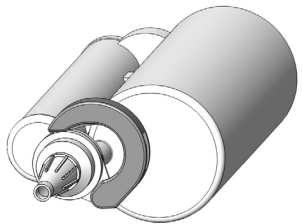

Lock the attached mixing tip by sliding the locking clip along the track until it encloses the mixing tip and reaches its final position. (Fig 4)

4

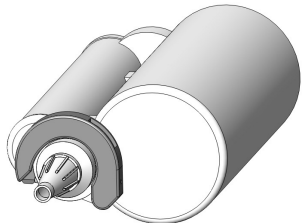Note: Always replace the mixing tip before each mixing procedure! The mixers are unable to press out impression material through material hardened in a used mixing tip.

Note: Make sure that the mixing tip sits firmly on the cartridge! Otherwise, the impression material will leak from the cartridge and mixing tip during extrusion, which may cause contamination.

Note: Prior to using a new cartridge for the first time, discard approx. the first 3 cm of the ejected material. After use, leave the used mixing tip on the cartridge as a seal.

Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

Warnings/precautions

- Keep out of the reach of children!
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Product performance characteristics

- The product complies with ISO 4823:2021, medium bodied consistency
- Mixing ratio of base/catalyst 5:1

Storage/disposal

- Store at 15 – 25 °C (59 – 77 °F) in a dry place.
- Do not use after the expiration date.
- Disposal must comply with national regulations.

Composition

SiO₂, vinyl/methyl silicones, hydrogen silicone, additives, pigments, catalysts.

Packaging

REF 9058MDR	1 Cartridge @ 380 ml 10 Mixing tips
-------------	--