»Soft relining primer«

de

Produktbeschreibung

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en-US

Product description

Soft relining primer is a single component dental bonding agent for use with permanently soft relining materials based on addition curing silicones (e.g. *VPS Soft relining material*).

Indications

Atrophy of the jaw and sharp alveolar ridge edges

Contraindications

Do not use the material where there are contact allergies or known allergies to any of the components.

Intended use

Bonding agent for permanently soft relining materials based on addition curing silicones.

Limitations of use

There are no known limitations of use.

Intended users

Dentist, dental technician

Patient target group

Persons receiving treatment as part of a dental procedure.

Recommended use

1. Use a brush to apply an even layer of *Soft relining primer* onto the denture, which has been prepared according to the manufacturer's instructions for the soft relining material used. When doing so, ensure that the edges are fully coated.
2. Allow the bonding agent to air-dry for a minute and apply at least one more layer in the same way.
3. The soft relining material based on addition curing silicones should then be processed according to the manufacturer's instructions.

Timing

Air-drying time	≥ 1 min
-----------------	---------

Notes for use

Close the bottle immediately after use to prevent evaporation of the solvent.

Interactions

It is important to ensure that the surface of the denture does not become contaminated by moisture, saliva, blood or monomer-containing resins both before and after the application of *Soft relining primer*.

Residual risks / side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

Warnings / precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

The product, together with soft relining material, complies with ISO 10139-2, 2016, Type A (soft).

Storage / disposal

- Store in a dry place at 15 to 25°C (59 to 77°F)! Do not store in the refrigerator or freezer.
- Protect against exposure to direct sunlight!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

Ethyl acetate (approx. 95 %), modified polyacrylate (approx. 3.5 %), additives

Packaging

REF 9109MDR	1 Bottle base @ 5 ml
-------------	----------------------