»One-component adhesive«

de

Produktbeschreibung

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en-us

Product description

One-component adhesive is a self-etching, light-curing, one-component bonding agent suitable for use on dentin and enamel for all adhesive restorations with light-curing composites, compomers and ormocers.

Indications

- Loss of tooth substance due to caries, trauma, tooth attrition/abrasion or development disorders
- Tooth or periodontal injuries which require composite splints

Contraindications

- Do not use directly on the exposed pulp.
- Do not use the material where there are contact allergies or known allergies to any of the components.

Intended use

Dentin and enamel bonding agent for all adhesive restorations with light-curing composites, compomers and ormocers.

Limitations of use

Application of *One-component adhesive* is contraindicated if a dry working area or the recommended application technique is not possible.

Intended users

Dentist

Patient target group

Persons receiving treatment as part of a dental procedure.

Recommended use

Carry out the cavity preparation, pulp protection and underfilling according to the manufacturer's instructions for use for the filling material to be applied. Clean the cavity using water spray and then dry using oil-free air. The dentin must not become too dry.

1. Add 2-3 drops of *One-component adhesive* to a mixing pad. Apply the bonding agent using an applicator brush, gently massage for 10 seconds onto the respective dentin and enamel surfaces and then allow the material to take effect for 20 seconds.
2. Blow and dry the material for at least 5 seconds to remove the solvent but ensure that sufficient material remains on the tooth.
3. Light-cure the bonding agent using a LED or halogen light-curing unit for 10 seconds. The light intensity should be a minimum of 600 mW/cm².
4. The filling material can be applied over the cured *One-component adhesive* immediately.

Notes for use

- Light-curing units should have an output of 450 nm and should be checked regularly. The light intensity should be a minimum of 600 mW/cm². Observe the manufacturer's instructions for the light-curing unit.
- Position the light-curing unit as close to the material as possible.

- Do not remove the oxygen inhibited layer because this is required for the bond with the next layer.
- When using One component adhesive in conjunction with other materials, follow the applicable manufacturer's instructions.
- If the material is applied into the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.
- The use of suitable pulp/dentin protection material to avoid irritation of the pulp is recommended. Cover areas near the pulp with a thin layer of a calcium hydroxide preparation. The surfaces not covered with calcium hydroxide are then treated with *One-component adhesive*.
- *One-component adhesive* is highly flammable and must therefore be kept away from potential ignition sources. Measures must be taken to prevent electrostatic risks. The vapors must not be inhaled.
- *One-component adhesive* contains methacrylates, which can cause skin and eye irritation. In the event of accidental contact with the eyes, rinse them with water and if necessary contact a doctor. In the event of unintended contact with skin, immediately wash thoroughly with water and soap.
- Ensure that the content of the bottle is not contaminated by a soiled brush.

Interactions

Materials containing eugenol can affect curing and can lead to the softening of the polymer content of *One-component adhesive*. Use of zinc oxide/eugenol cements in combination with *One-component adhesive* should therefore be avoided.

Residual risks / side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

Warnings / precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

SBS bonding values on non-etched tooth substance measured in accordance with ISO29022:2013 are ≥ 15 MPa.

»One-component adhesive«

Storage / disposal

- Store in a dry place in the refrigerator at 2 – 8 °C/36 – 46 °F.
- Close bottles immediately after use.
- Do not use the material past the stipulated shelf life or if the fluid assumes a gelatinous consistency.
- Disposal must comply with national regulations.

Composition

Resin matrix based on bis-GMA, HEMA (approx. 65 %), water/ethanol (approx. 33 %), additives

Packaging

REF 2055MDR	1 Bottle @ 5ml 50 Disposable brushes white 1 Mixing pad
-------------	---