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Instructions for use »One-component adhesive«

One-component adhesive is a light-curing, self-etching one component adhesive system. It can be used for all restorations with light-curing composites, compomers and ormoceres.

Recommended use

Prepare cavity, pulp protection and base lining according to the instructions for use of the filling material. Clean cavity with water spray and dry with oil-free air. Dentin must not be overdried!

1. Put 2-3 drops of *One-component adhesive* on a mixing pallet. Apply on tooth with single-use brush and gently work into tooth structure for 10 sec. Let material set for more 20 sec.
2. Blow and dry material at least 5 sec. to remove solvent. Pay special attention to still have sufficient material on tooth.
3. Light cure adhesive with LED or halogen curing unit for 10 sec.
4. Filling material may be applied onto cured *One-component adhesive* immediately.

Please note

- A suitable pulp / dentin protection is to be used to prevent irritation of pulp. Areas close to pulp have to be covered with a thin layer of a suitable underfilling such as calcium hydroxide. Areas, which are not covered with calcium hydroxide, are to be treated with *One-component adhesive*.
- *One-component adhesive* is easily inflammable. Keep away from possible ignition sources. Measures against electrostatic charge have to be taken. Fume of the product should not be inhaled.
- *One-component adhesive* contains methacrylate, which can lead to skin and eye irritations. In case of contact with eyes flush eyes with a lot of water and if necessary seek medical attention.
- Contamination of the content of the bottle caused by contaminated single-use brushes is to be avoided.

Interactions

Materials containing eugenol can inhibit polymerization of *One-component adhesive*. Therefore do not use zinc-oxide cements in connection with *One-component adhesive*.

Contraindications

Application is contraindicated for direct application on the opened pulp or when allergies against any components of *One-component adhesive* are known.

Side effects

Side effects are not known to date. In singular cases, hypersensitivity to components of the material cannot be excluded.

Composition

One-component adhesive contains BIS-GMA based dental resins, ethanol, water, additives and catalysts.

Classification

No relevant norm.

Storage

Store dry and cool between 2°C - 8°C/ 36 - 46 °F. Avoid direct sunlight. Please close bottle immediately after use. Do not use after the expiration date.

Available product sizes

One-component adhesive

REF 2055 1 Bottle @ 5ml

50 Single-use brushes white

1 Mixing pallet

Keep away from children! For dental use only!