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## »Self etching pit and fissure sealant«

en-US

### Product description

*Self etching pit and fissure sealant* is a highly flowable, light-curing composite for sealing fissures.

### Indications

Molars and premolars with pronounced deep fissures and increased caries risk.

### Contraindications

- Do not apply the material to the exposed pulp.
- Do not use the material if a patient has contact allergies or known allergies to any of the ingredients.

### Intended purpose

- Fissure sealing
- Extended fissure sealing

### Limitations of use

- Application of the material is contraindicated if dry isolation or the recommended application technique is not possible.
- Do not use the material on etched dentin

### Intended users

Dentist

### Patient target group

People treated in the course of a dental procedure.

### Recommended use

1. For extended fissure sealing, widen particularly narrow and vial-shaped fissures.
2. Clean enamel with fluoride-free paste, remove all cleaning residue with water spray and dry the tooth with water-free and oil-free air. In doing so, leave the enamel surface moist.

**Note:** No special separate etching and bonding step is required for the use of *Self etching pit and fissure sealant*. However, prior to application, contaminants such as saliva, blood, eugenol or other substances must be removed from the surface of the tooth, since these substances can impair curing or adhesion. This can be achieved, for example, by etching enamel for 15–30 seconds with 37% phosphoric acid gel, rinsing for 15 seconds with water, and drying with oil-free and water-free air. In doing so, leave the enamel surface moist.

3. Press the syringe to apply the material onto the surface of the fissure or cavity wall with the aid of the Luer-Lock-Tips and use the brush to massage a thin layer ( $\approx 0.5$  mm) into the entire surface for 25 seconds.
4. Remove any excess, if necessary.
5. Light-cure for 20 seconds.
6. Where necessary, use the material to build up layers with a maximum 2 mm layer thickness and light-cure for 20 seconds.

**Note:** It is not necessary to massage the individual layers more than once.

7. Check the occlusion and correct if necessary.

### Notes for use

- If the material is applied into the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.
- Light-curing units should have an output of 450 nm and should be checked regularly. The light intensity should be at least 600 mW/cm<sup>2</sup>. Observe the manufacturer's instructions for the light-curing unit.
- Position the light-curing unit as close to the material as possible.
- Do not remove the oxygen inhibited layer because this is required for the bond with the next layer.
- When using *Self etching pit and fissure sealant* in conjunction with other materials, follow the applicable manufacturer's instructions

### Interactions

- Materials containing eugenol, moisture and oily air may inhibit polymerization at the contact area. Avoid materials containing eugenol, moisture and oily air!
- The use of cationic oral rinses, plaque indicators and chlorhexidine may lead to discoloration.

### Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

### Warnings / precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse the eyes thoroughly with plenty of water and consult a physician if necessary.
- The brush holder is reusable after disinfection, although it should be replaced if visible signs of wear, e.g. scratches, appear. Disinfection can be carried out using a commercially available wiping or dipping disinfectant. It is advisable to only use disinfecting solutions listed by the RKI (Robert Koch Institute).
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

### Performance characteristics of the device

Complies with ISO 6874:2015

Polymerization depth  $\geq 4$  mm with a 20 second light-curing time when the light intensity of the lamp is  $\geq 600$  mW/cm<sup>2</sup>

### Storage / disposal

- Store in a dry place (15-25°C/59-77°F) at room temperature!

## »Self etching pit and fissure sealant«

- Reseal syringes after use!
- Do not use after the expiration date!

### Composition

Dental glass (approx. 50 %), resin matrix based on EBPADMA, UDMA, TED-MA (approx. 49 %), additives, pigments, catalysts

### Packaging

REF 2105MDR	2 Syringes @ 1 ml; shade opaque white 20 Luer-Lock-Tips 20 Brushes 1 Brush holder
-------------	--